

Twin Stim® Plus 3rd Edition Combo Stimulator

INSTRUCTION MANUAL



**This manual is valid for the
Twin Stim® Plus 3rd Edition Combo Stimulator**

This user manual is published by Compass Health Brands.

Compass Health Brands does not guarantee its contents and reserves the right to improve and amend it at any time without prior notice. Amendments may however be published in new editions of this manual.

All Rights Reserved. Rev. B © 2023



United States Federal Law restricts this device to sale by or on the order of a physician or licensed practitioner

Conformity to safety standards

Compass Health Brands declares that the device complies with following normative documents:

**IEC60601-1, IEC60601-1-2, IEC60601-1-11, IEC60601-2-10,
ISO10993-1, ISO10993-5, ISO10993-10, ISO14971**

TABLE OF CONTENTS

Chapter 1: GENERAL DESCRIPTION.....	4
Chapter 2: INTRODUCTION.....	5
Chapter 3: IMPORTANT SAFETY INFORMATION.....	8
Chapter 4: CONSTRUCTION.....	12
Chapter 5: TECHNICAL SPECIFICATIONS.....	15
Chapter 6: THERAPY PROGRAM AND WAVEFORMS..	17
Chapter 7: UNIT AND ACCESSORIES.....	24
Chapter 8: OPERATING INSTRUCTIONS.....	25
Chapter 9: PARAMETER CONTROLS.....	29
Chapter 10: PATIENT COMPLIANCE METER.....	32
Chapter 11: MAINTENANCE AND STORAGE.....	33
Chapter 12: TROUBLESHOOTING.....	34
Chapter 13: NORMALIZED SYMBOLS.....	36
Chapter 14: WARRANTY.....	37

Chapter 1: GENERAL DESCRIPTION

The Twin Stim® Plus 3rd Edition Combo Stimulator is a battery operated pulse generator that sends electrical impulses electrodes to the body and reach the nerves and underlying muscle group. This unit is a combination stimulator of TENS, EMS, IF (Interferential) and RUSSIAN which can be used for muscle stimulation and pain relief. The device is provided with four controllable output channels, each independent of each other. An electrode pair can be connected to each output channel. The intensity level is controlled by press buttons.

Chapter 2: INTRODUCTION

EXPLANATION OF PAIN

Pain is a warning system and the body's method of telling us that something is wrong. Pain is important; without it abnormal conditions may go undetected, causing damage or injury to vital parts of our bodies. Even though pain is a necessary warning signal of trauma or malfunction in the body, nature may have gone too far in its design. Aside from its value in diagnosis, long-lasting persistent pain serves no useful purpose. Pain does not begin until coded message travels to the brain where it is decoded, analyzed, and then reacted to. The pain message travels from the injured area along the small nerves leading to the spinal cord. Here the message is switched to different nerves that travel up the spinal cord to the brain. The pain message is then interpreted, referred back and the pain is felt.

EXPLANATION OF TENS

Transcutaneous Electrical Nerve Stimulation is a non-invasive, drug free method of controlling pain. TENS uses tiny electrical impulses sent through the skin to nerves to modify your pain perception. TENS does not cure any physiological problem; it only helps control the pain. TENS does not work for everyone; however, in most patients it is effective in reducing or eliminating the pain, allowing for a return to normal activity.

HOW TENS WORKS

There is nothing "magic" about Transcutaneous Electrical Nerve Stimulation (TENS). TENS is intended to be used to relieve pain. The TENS unit sends comfortable impulses through the skin that stimulate the nerve (or nerves) in the treatment area. In many cases, this stimulation will greatly reduce or eliminate the pain sensation the patient feels. Pain relief varies by individual patient, mode selected for therapy, and the type of pain. In many patients, the reduction or elimination of pain lasts longer than the actual period of stimulation (sometimes as much as three to four times longer). In others, pain is only modified while stimulation actually occurs. You may want to discuss this therapy option with your physician or therapist to see if it will benefit your pain management needs.

EXPLANATION OF EMS

Electrical Muscle Stimulation is an internationally accepted and proven way of treating muscular injuries. It works by sending electronic pulses to the muscle needing treatment; this causes the muscle to exercise passively. It is a product derived from the square waveform, originally invented by John Faraday in 1831. Through the square wave pattern it is able to work directly on muscle motor neurons. The Twin Stim® Plus 3rd Edition Combo Stimulator has low frequency and this in conjunction with the square wave pattern allows direct work on muscle groupings. This is being widely used in hospitals and sports clinics for the treatment of muscular injuries and for the re-education of paralyzed muscles, to prevent atrophy in affected muscles and improving muscle tone and blood circulation.

HOW EMS WORKS

The EMS units send comfortable impulses through the skin that stimulate the nerves in the treatment area. When the muscle receives this signal it contracts as if the brain has sent the signal itself. As the signal strength increases, the muscle flexes as in physical exercise. Then when the pulse ceases, the muscle relaxes and the cycle starts over again, (Stimulation, Contraction and Relaxation.) Powered muscle stimulators should only be used under medical supervision for adjunctive therapy for the treatment of medical diseases and conditions.

EXPLANATION OF IF (Interferential)

Interferential Stimulation (IF) is an anti-inflammatory based treatment modality. Interferential stimulation is characterized by two alternating-current sine waves or square waves of differing frequencies that “work” together to produce an interferential current that is also known as a beat pulse or alternating modulation frequency. One of the two currents is usually held at 4,000 Hz, and the other can be held constant or varied over a range of 4,001 to 4,150 Hz. Because of the frequency, the interferential wave meets low impedance when crossing the skin to enter deep into soft tissues. The interferential currents reportedly can stimulate sensory, motor, and pain fibers. These large impulse fibers interfere with the transmission of pain messages at the spinal cord level. This deep tissue penetration stimulates parasympathetic nerve fibers for increased blood flow and edema reduction. It utilizes the low electric-current to

stimulate muscle nerves to achieve the symptomatic relief of chronic intractable pain, post-traumatic pain, and post-surgical pain.

EXPLANATION OF RUSSIAN

Russian stimulation uses medium frequencies to provide electrical stimulation to muscle groups and is used to reduce muscle spasms as well as for muscle strengthening. Russian stimulation is a specific form of electro-stimulation with a Symmetrical Biphasic Square waveform produced by dividing a 2500Hz carrier frequency into 20~80Hz packets. This method was claimed by its author (Kots) to produce maximal muscle strengthening effects without significant discomfort to the patient.

INDICATION FOR USE

For TENS and IF (Interferential):

- ▶ Symptomatic relief of chronic intractable pain
- ▶ Post traumatic pain
- ▶ Post surgical pain

For EMS and Russian:

- ▶ Relaxation of muscle spasms
- ▶ Increase of local blood flow circulation
- ▶ Prevention or retardation of disuse atrophy
- ▶ Muscle re-education
- ▶ Maintaining or increasing range of motion
- ▶ Immediate post surgical stimulation of muscles to prevent venous thrombosis

Chapter 3:

IMPORTANT SAFETY INFORMATION

Read instruction manual before operation. Be sure to comply with all “Contraindications”, “Warnings”, “Cautions” and “Adverse reactions” in the manual. Failure to follow instructions can cause harm to user or device.

3.1 Contraindications

- 1) This device should not be used for symptomatic local pain relief unless etiology is established or unless a pain syndrome has been diagnosed.
- 2) This device should not be used when cancerous lesions are present in the treatment area.
- 3) Stimulation should not be applied over swollen, infected, inflamed areas or skin eruptions (e.g. phlebitis, thrombophlebitis, varicose veins, etc.).
- 4) Electrodes must not be applied to sites that might cause current/ stimulation to flow through the carotid sinus region (anterior neck) or trans-cerebrally (through the head).
- 5) Do not use this device if the patient has a demand-type cardiac pacemaker or any implanted defibrillator.
- 6) This device should not be used over poorly enervated areas.
- 7) This device should not be used on patients with epilepsy.
- 8) This device should not be used on patients with serious arterial circulatory problems in the lower limbs.
- 9) This device should not be used on patients with abdominal or inguinal hernia.
- 10) Do not use this device if you have heart disease without consulting your physician.

3.2 Warnings, cautions and adverse reactions



Warnings

- 1) This device should be used only under the continued supervision of a licensed physician.
- 2) The long-term effects of chronic electrical stimulation are unknown. Electrical stimulation devices do not have any curative value.

- 3) TENS is a symptomatic treatment and, as such, suppresses the sensation of pain, which would otherwise serve as a protective mechanism.
- 4) Safety has not been established for the use of therapeutic electrical stimulation during pregnancy. Do not use during pregnancy unless directed by your physician.
- 5) Electrical stimulation is not effective for pain of central origin.
- 6) Electronic monitoring equipment (such as ECG monitors and ECG alarms) may not operate properly when electrical stimulation is in use.
- 7) Stimulation should not be applied over the carotid sinus nerves, particularly in patients with a known sensitivity to the carotid sinus reflex.
- 8) Stimulation should not be applied over the anterior neck or mouth. Severe spasm of the laryngeal and pharyngeal muscles may occur and the contractions may be strong enough to close the airway or cause difficulty in breathing.
- 9) Stimulation should not be applied transthoracically in that the introduction of electrical current into the heart may cause cardiac arrhythmias.
- 10) Stimulation should not take place while the user is connected to high-frequency surgical equipment, it may cause burn injuries on the skin under the electrodes, as well as problems with the stimulator.
- 11) Do not use the stimulator in the vicinity of shortwave or microwave therapy equipment, since this may affect the output power of the stimulator.
- 12) Never use in environments with high humidity such as in the bathroom or when having a bath or shower.
- 13) Caution should be used in applying electrical stimulation to patients suspected of having heart disease. Further clinical data is needed to show there are no adverse results.
- 14) Never use near the heart. Stimulation electrodes should never be placed anywhere on the front of the thorax (marked by ribs and breastbone), but above all not on the two large pectoral muscles. Here it can increase the risk of ventricular fibrillation and lead to cardiac arrest.
- 15) Electrodes should not be placed over the eyes, in the mouth, near the genitals or internally.
- 16) Never use on the areas of the skin which lack normal sensation.
- 17) Apply the electrodes to clean, dry, and unbroken skin only.
- 18) Keep electrodes separate during treatment, electrodes in contact with other could result in improper stimulation or skin burns.
- 19) Keep the stimulator out of reach of children.
- 20) Consult your doctor if you are in any doubt whatsoever.



Cautions:

- 1) Federal law (USA) restricts this device to sale by or on the order of a physician.
- 2) For single patient use only.
- 3) Keep yourself informed of the contraindications.
- 4) This stimulator is not intended for unattended, personal use by patients who have noncompliant, emotionally disturbed, dementia, or low IQ.
- 5) Read, understand, and practice the warnings, cautions and operating instructions. Know the limitations and hazards associated with using any device. Observe the precautionary and operational decals placed on the unit. Always follow the operating instructions prescribed by your healthcare practitioner.
- 6) The instruction of use was listed; any improper use may be dangerous.
- 7) Do not use this device for undiagnosed pain syndromes until consulting a physician.
- 8) Patients with an implanted electronic device, such as a cardiac pacemaker, implanted defibrillator, or any other metallic or electronic device should not use this device without first consulting a doctor.
- 9) Stimulation delivered by this device may be sufficient to cause electrocution. Electrical current of this magnitude must not flow through the thorax or across the chest because it may cause a cardiac arrhythmia.
- 10) Do not place electrodes on the front of the throat as spasm of the Laryngeal and Pharyngeal muscle may occur. Stimulation over the carotid sinus (neck region) may close the airways, make breathing difficult, and may have adverse effects on the heart rhythm or blood pressure.
- 11) Do not place electrodes on your head or at any sites that may cause the electrical current to flow trans-cerebrally (through the head).
- 12) Patients with heart disease, epilepsy, cancer or any other health condition should not use this device without first consulting a physician.
- 13) Some patients may experience skin irritation or hypersensitivity due to the electrical stimulation or silicone rubber. If rash develops or pain persists, discontinue use and consult a doctor.
- 14) Electrode placement and stimulation settings should be based on the guidance of prescribing practitioner.
- 15) Effectiveness is highly dependent upon patient selection by a person qualified in the management of pain afflicted patients.

- 16) Isolated cases of skin irritation may occur at the site of the electrode placement following long-term application. If this occurs, discontinue use and consult your physician.
- 17) The electrodes are only to be placed on healthy skin. Avoid skin irritation by ensuring that good contact is achieved between electrodes and skin.
- 18) If the stimulation levels are uncomfortable or become uncomfortable, reduce the stimulation Intensity to a comfortable level and contact your physician if problems persist.
- 19) This device should not be used while driving, operating machinery, close to water, or during any activity in which involuntary muscle contractions may put the user at undue risk of injury.
- 20) Never use the device in rooms where aerosols (sprays) are used or pure oxygen is being administered.
- 21) Do not use it near any highly flammable substances, gases or explosives.
- 22) Do not use this device at the same time as other equipment which sends electrical pulses to your body.
- 23) Do not confuse the electrode cables and contacts with your headphones or other devices, and do not connect the electrodes to other devices.
- 24) Do not use sharp objects such as pencil point or ballpoint pen to operate the buttons on the control panel.
- 25) Inspect Applicator cables and associated connectors before each use.
- 26) Turn the device off before applying or removing electrodes.
- 27) Electrical stimulators should be used only with the leads and electrodes recommended for use by the manufacturer.
- 28) This device has no AP/APG protection. Do not use it in the presence of explosive atmosphere and flammable mixture.


Adverse reactions:


Skin irritation from the electrode gel and electrode burns are potential adverse reactions. If skin irritation occurs, discontinue use and consult your physician.

Note: Always use electrodes that are legally marketed and sold in the United States under 510K guidelines.

Chapter 4: CONSTRUCTION


PANEL

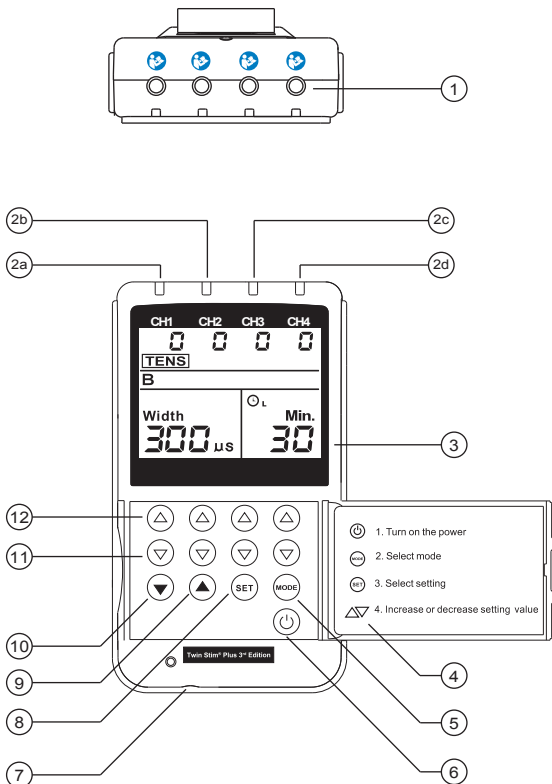

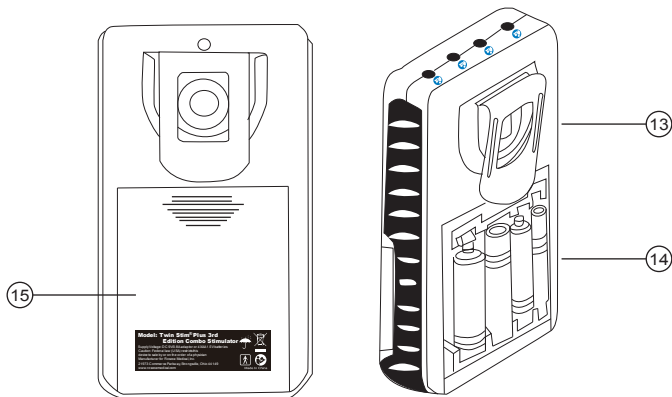

- 1) Lead connector
- 2) Channel 1, Channel 2, Channel 3 and Channel 4
- 3) Liquid crystal display
- 4) Panel cover
- 5) Mode control
- 6) Power on/off control
- 7) AC Adapter
- 8) Set control
- 9) Increment control – for Pulse Rate, Pulse Width and time indicators, interference frequency
- 10) Decrement control – for Pulse Rate, Pulse Width and time indicators, interference frequency
- 11) Intensity decrease control
- 12) Intensity increase control
- 13) Belt clip
- 14) Battery case
- 15) Battery cover

LCD

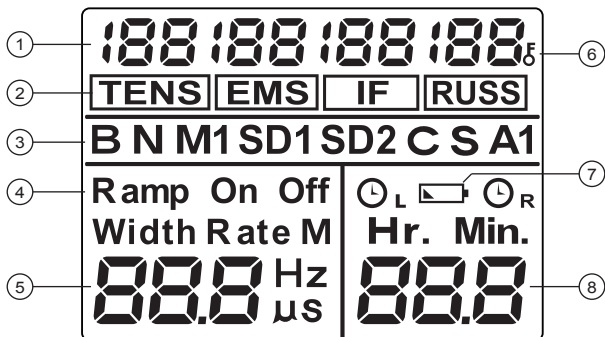

- 1) Output intensity from channel 1 to channel 4
- 2) TENS, EMS, IF (Interferential) and RUSS (Russian) mode
- 3) Therapy program
- 4) Time symbol
- 5) Pulse rate and pulse width indicator
- 6) Lock symbol
- 7) Low battery symbol
- 8) Time indicator

Chapter 5: TECHNICAL SPECIFICATIONS

The technical specification details of Twin Stim® Plus 3rd Edition Combo stimulator are as follows:

No.	MECHANISM	TECHNICAL DESCRIPTION
01	Channel	Four, isolated between channels
02	Output intensity	0~40Vpp (at 500 ohm load)
03	Waveform	Mono/Bi-Phasic Square Pulse
04	Power source	DC 6V, 4×AA 1.5V battery, or AC Adapter output :DC 9V,800mA
05	Size	13.8cm(L)x7.8cm(W)x2.8cm(H)
06	Weight	425 grams with battery.
07	Pulse Rate	TENS/EMS: 2Hz~150Hz; IF: 4000Hz~4150 Hz ; RUSS: fixed rate:2500 Hz; Burst rate:20Hz~80Hz
08	Pulse width	50μs~300μs
09	On Time	Adjustable, 2~90 seconds,1Sec./step
10	Off Time	Adjustable, 0~90 seconds,1Sec./step
11	Ramp Time	Adjustable, 1~8 seconds, 1 Sec./ step, The “On” time will increase and decrease in the setting value.
12	Treatment time	Two adjustable timers, from 1 to 60 minutes or Continuous. Adjustable in 1 minute each step from 1 to 15 minutes, and 5 minutes each step from 15 to 60 minutes. Treatment time countdown automatically.

13	Patient Compliance Meter	This unit can store 60 sets of operation records. Total recorded time is 999 hours.
14	Low battery symbol	A low battery symbol will show up when the battery is low.
15	Operating conditions	Temperature: 0°C ~40°C Relative Humidity: 30%~75% Atmosphere Pressure: 700hPa~1060hPa
16	Storage conditions	Temperature:-10°C~50°C Relative humidity:10%~90% Atmospheric pressure: 700hPa~1060hPa
17	Service life of the device	3 years
18	Service life of the batteries	With new batteries (4xAA,Peak battery), approx. 60 days when used for 30 minutes a day in TENS B mode at the 1/3 of maximum step.

Chapter 6:

THERAPY PROGRAM AND WAVEFORMS


TENS PROGRAM

No.	MECHANISM	TECHNICAL DESCRIPTION
01	Program	Five TENS program: B(Burst), N(Normal), M(Modulation),SD1(Strength Duration 1), SD2 (Strength Duration 2)
02	Burst program (B)	Burst rate: Adjustable, 0.5 ~ 5Hz Pulse width adjustable, 50~300µs Frequency fixed = 100 Hz
03	Normal program (N)	The pulse rate and pulse width are adjustable. It generates continuous stimulation based on the setting value.
04	Modulation program (M)	Modulation mode is a combination of pulse rate and pulse width modulation. The pulse rate and width are automatically varied in a cycle pattern. The pulse width is decreased by 50% from its original setting in 0.5s, and then the pulse rate is decreased by 50% from its original setting in 0.5 seconds. Total cycle time is 1 second. In this mode, pulse rate (2~150Hz) and pulse width (50~300µs) are fully adjustable.
05	Strength-Duration program (SD1)	The SD1(Strength-Duration) mode consists of automatic modulation intensity and pulse width in 40% range. The intensity is always increasing while the pulse width is decreasing and vice-versa. The intensity is decreased by 40% while the pulse width is increased by 40% in 5 seconds. In the next 5 seconds, the intensity is increased by 40% while the pulse width is decreased by 40%. Total cycle time is 10


		seconds. Pulse rate (2~150Hz) and pulse width (50~300 μ s) are fully adjustable.
06	Strength-Duration program (SD2)	The SD2(Strength-Duration) mode consists of automatic modulation intensity and pulse width in 70% range. The intensity is always increasing while the pulse width is decreasing and vice-versa. The intensity is decreased by 70% while the pulse width is increased by 70% in 5 seconds. In the next 5 seconds, the intensity is increased by 70% while the pulse width is decreased by 70%. Total cycle time is 10 seconds. Pulse rate (2~150Hz) and pulse width (50~300 μ s) are fully adjustable.

TENS WAVEFORMS


1. Burst

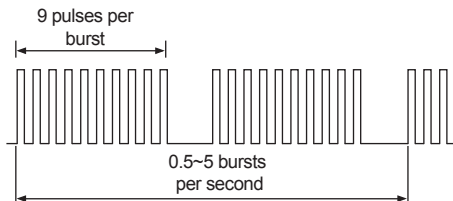


2. Normal

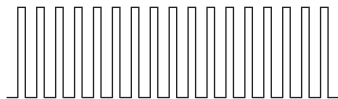


3. Modulation

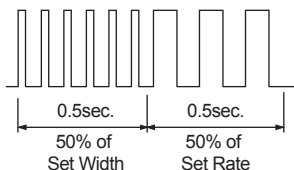

4. SD1(Strength-Duration)

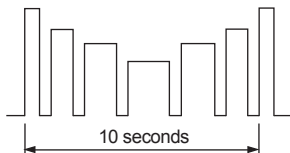

5. SD2(Strength-Duration)

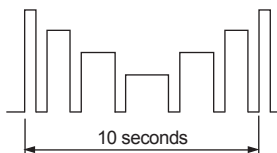

EMS PROGRAM

No.	MECHANISM	TECHNICAL DESCRIPTION
01	Program	Three EMS Modes: C(Constant), S(Synchronous), A(Alternate)
02	Constant program (C)	Constant stimulation based on setting value. Only pulse width, pulse rate and timer are adjustable in this mode. "Constant" is equal to the "Normal" mode in TENS.


03	Synchronous program (S)	Stimulation of both channels occurs synchronously. The "ON" time including "Ramp up" and "Ramp down" time. Therefore, the setting of ON time should be no less than two times of the "Ramp" time in this mode. ON time $\geq$ Ramp up + Ramp down
04	Alternate program (A)	The stimulation of the CH2 will occur after the 1 st contraction of CH1 is completed. In this mode, the setting of ON time should be no less than two times of the "Ramp" time. The OFF time should be equal or more than the ON time. ON time $\geq$ Ramp up + Ramp down OFF time $\geq$ ON time

EMS WAVEFORMS


Constant (C)



Synchronous (S)

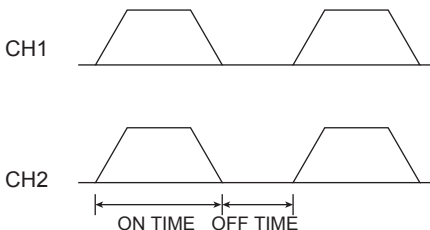

Alternate (A)

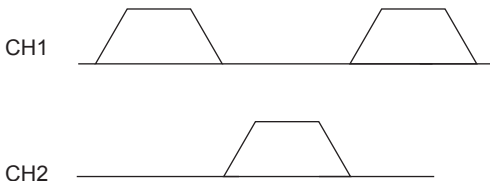


IF (Interferential) PROGRAM

No.	MECHANISM	TECHNICAL DESCRIPTION
01	Program	Three IF Modes: N (Normal), M (Modulation) and M1 (Modulation 1)
02	Normal program (N)	The pulse rates of the CH1 and CH3 are fixed in 4000Hz; The interference frequencies of CH2 and CH4 are adjustable from 1Hz to 150Hz. The pulse rates of CH2 and CH4 are based on the setting value and fixed frequency (4000Hz), means that pulse rate=4000Hz+Interference frequency.
03	Modulation program (M)	The pulse rates of the CH1 and CH3 is fixed in 4000Hz; The pulse rates of CH2 and CH4 are based on the interference frequencies and fixed frequency (4000Hz), means that pulse rate = 4000Hz+Interference frequency. The interference frequencies are adjustable from 1Hz to 80Hz. The pulse rates of CH2 and CH4 are modulated between setting value and 4001Hz in 10 seconds.

04	Modulation program (M1)	The pulse rates of the CH1 and CH3 is fixed in 4000Hz; The pulse rates of CH2 and CH4 are based on the interference frequencies and fixed frequency (4000Hz), means that pulse rate = 4000Hz+Interference frequency. The interference frequencies are adjustable from 80Hz to 150Hz. The pulse rates of CH2 and CH4 are modulated between setting value and 4080Hz in 10 seconds.
----	-------------------------	---

IF (Interferential) WAVEFORM

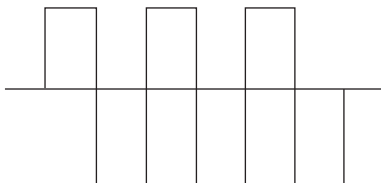


RUSS (Russian) PROGRAM

No.	MECHANISM	TECHNICAL DESCRIPTION
01	Program	Three RUSS Modes: N (Normal), M (Modulation) and M1 (Modulation 1)
02	Normal program (N)	Constant stimulation based on setting value. Burst frequency is adjustable from 20Hz to 80 Hz. The duty cycle keep at 50%.
03	Modulation program (M)	Duty cycle is automatically varied in a cycle time. The duty cycle is increase from 10% to 50% in 10 seconds, and then decrease from 50% to 10% in next 10 seconds. In this program, burst frequency is adjustable from 20Hz to 80Hz.

04	Modulation program (M1)	Burst frequency is decreased from the setting value to 20Hz in 10 seconds, and then increased from 20Hz to the setting value in next 10 seconds. In this program, the burst frequency is adjustable from 20Hz to 80Hz.
----	-------------------------	--

RUSSIAN WAVEFORM

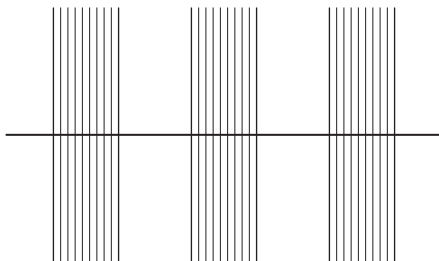

Chapter 7: ACCESSORIES

Twin Stim® Plus 3rd Edition Combo Stimulator comes complete with standard accessories as below:


No	DESCRIPTION	QTY'
1	50 X 50 mm Adhesive Electrodes	8PCS
2	Lead wires	4PCS
3	AA Battery	4PCS
4	Instruction Manual	1PC
5	Carrying Case	1PC
6	AC Adapter	1PC

Chapter 8: OPERATING INSTRUCTIONS

8.1 Check/Replace the batteries

Over time, in order to ensure the functional safety of device, changing the batteries is necessary.

- 1) Slide the battery cover and open.
- 2) Insert four AA batteries into the battery compartment.
- 3) Make sure you are installing the batteries properly.
Be sure to match the positive and negative ends of the battery to the marking in the battery compartment of the device.
- 4) Replace the battery cover and press to close.

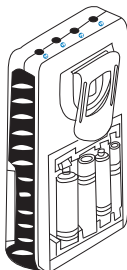

Caution:


- ▶ Remove the batteries if the device is not in use for long periods of time.
- ▶ Do not mix old and new batteries or difference type of batteries.
- ▶ Warning: If batteries leak and come into contact with the skin or eyes, wash immediately with copious amounts of water.
- ▶ Batteries must be handled by an adult. Keep batteries out of the reach of children.
- ▶ Remove exhausted batteries from the unit.
- ▶ Dispose of batteries safety according to local regulation.

8.2 Connect electrodes to lead wires

Insert the lead wire connector into electrodes connector (standard 0.08 inch female connection).
Make sure no bare metal of the pins is exposed.

Caution:


Always use the electrodes with CE mark, or which are legally marketed in the US under 510(K) procedure.

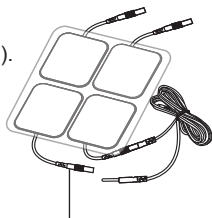

Connection Cables

8.3 Connect lead wires to device

- 1) Before proceeding to this step, be sure the device is completely turned OFF.
- 2) Insert the lead wires into the lead connectors on top of the device.

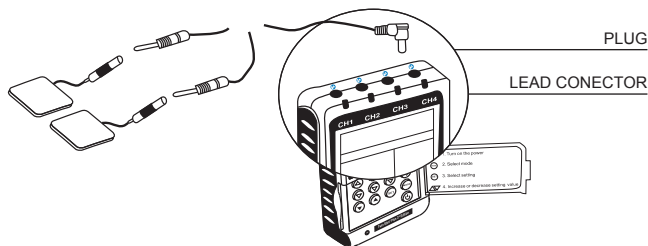

⚠ Caution:

Do not insert the plug of the patient lead wire into any AC power supply socket.

8.4 Electrode

8.4.1 Electrode options

The electrodes are disposable and should be routinely replaced when they start to lose their adhesive nature. If you are unsure of your electrode adhesive properties, order replacement electrodes. Replacement electrodes should be re-ordered through or on the advice of your physician to ensure proper quality. Follow application procedures outlined in electrode packing, to maintain optimal stimulation and to prevent skin irritation.

8.4.2 Place electrodes on skin

Apply electrodes to the exact site indicated by your physician or therapist, before applying electrodes, be sure the skin surface over which electrodes are placed is thoroughly cleaned and dried. Make sure the electrodes are placed firmly to the skin and make good contact between the skin and the electrodes. Place the electrodes over the skin; attach them properly, firmly, and evenly.

⚠ Caution:

- 1) Before applying the self-adhesive electrodes, it is recommended to wash

- and degrease the skin, and then dry it.
- 2) Do not turn on the device when the self-adhesive electrodes are not positioned on the body.
 - 3) Never remove the self-adhesive electrodes from the skin while the device is still turned on.
 - 4) It is recommended that, at a minimum, 4cm x 4cm self-adhering based, square electrodes are used at the treatment area.

8.4.3 Electrode placement

The placement of electrodes can be one of the most important parameters in achieving success with therapy. Of utmost importance is the willingness of the physician to try the various styles of electrode placement to find which method best fits the needs of the individual patient.

Every patient responds to electrical stimulation differently and their needs may vary from the conventional settings suggested here. If the initial results are not positive, speak to your physician about alternative stimulation settings and/or electrode placements. Once an acceptable placement has been achieved, mark down the electrodes sites and the settings, so the patient can easily continue treatment at home.

8.5 Turn on

Before using the device for the first time, you are strongly advised to take careful note of the counter-indications and safety measures detailed at the beginning of this manual (Safety information), as this powerful equipment is neither a toy nor a gadget!

Press the [] button to turn on the device.


8.6 Select the therapeutic mode and program

There are 4 therapeutic modes available – TENS, EMS, IF and RUSS.

The therapeutic mode and program can be selected by pressing the [MODE] button.

The LCD displays as

(TENS)→B; (TENS)→N; (TENS)→M; (TENS)→SD1; (TENS)→SD2;
 (EMS)→C; (EMS)→S; (EMS)→A;
 (IF)→N; (IF)→M; (IF)→M1;
 (RUSS)→N; (RUSS)→M; (RUSS)→M1.

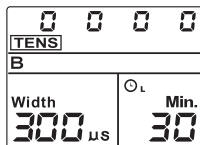

8.7 Adjust output intensity

There are 4 sets of intensity buttons, which correlate to each of the 4 channels. Slowly increase or decrease the intensity as directed by your physician by pressing the up or down intensity buttons under the respective channel.

Press the up arrow [Δ] button to increase intensity and press the down arrow [∇] button to decrease intensity.



Caution:

If the stimulation levels are uncomfortable or become uncomfortable, reduce the stimulation intensity to a comfortable level and contact your medical practitioner if problems persist.

8.8 Turn off the device

After treatment, turn the device off by pressing the [⏻] button.

Chapter 9: PARAMETER CONTROLS

There are 4 therapeutic modes available –TENS, EMS, IF and RUSS. Each mode's parameters can be set by pressing [SET] button to enter setting mode, and then adjust the parameters by pressing [▲] or [▼] button.

9.1 Steps to set a new TENS program

9.1.1 Select TENS therapeutic mode and program

Press [MODE] button until the LCD display “TENS” and expected program (B, N, M, SD1, SD2).

9.1.2 Set pulse width

Press [SET] button to enter setting mode, the “Width” indicator displays on LCD, press [▲] or [▼] button to adjust the parameter from 50 μ s to 300 μ s.

9.1.3 Set pulse rate

Press [SET] button again, The “Rate” indicator displays on LCD, press [▲] or [▼] button to adjust the parameter from 2Hz to 150Hz. In Burst (B) program, you can set the burst rate from 0.5Hz to 5Hz.

9.1.4 Set timer

There are two adjustable timers available. The left timer controlling the treatment time of CH1 and CH2. The right timer controlling the treatment time of CH3 and CH4. The treatment time is adjustable from 1 to 60 minutes or C (Continuous). Press [SET] button to enter this menu, then press [▲] or [▼] button to adjust the setting. Press [▲] button when the timer shows 60 minutes, it will be switched to continuous stimulation. Two timers can be set in the same way.

9.2 Steps to set a new EMS program

9.2.1 Select EMS therapeutic mode and program

Press [MODE] button until the LCD display “EMS” and expected program (C, S, A).

9.2.2 Set ramp time (A or S program)

Press [SET] button to enter setting mode, The “Ramp” indicator displays on LCD when you select Alternate (A) or Synchronous (S) program, press

[▲] or [▼] button to adjust the parameter from 1s to 8s.

9.2.3 Set ON time (A or S program)

Press [SET] button again, The "On" indicator displays on LCD, press [▲] or [▼] button to adjust the parameter from 2s to 90s.

9.2.4 Set OFF time (A or S program)

Press [SET] button again, The "Off" indicator displays on LCD, press [▲] or [▼] button to adjust the parameter from 2s to 90s.

9.2.5 Set pulse width

Press [SET] button again, the "Width" indicator displays on LCD, press [▲] or [▼] button to adjust the parameter from 50 μ s to 300 μ s.

9.2.6 Set pulse rate

Press [SET] button again, The "Rate" indicator displays on LCD, press [▲] or [▼] button to adjust the parameter from 2Hz to 150Hz.

9.2.7 Set timer

There are two adjustable timers available. The left timer controlling the treatment time of CH1 and CH2. The right timer controlling the treatment time of CH3 and CH4. The treatment time is adjustable from 1 to 60 minutes or C (Continuous). Press [SET] button to enter this menu, then press [▲] or [▼] button to adjust the setting. Press [▲] button when the timer shows 60 minutes, it will be switched to continuous stimulation. Two timers can be set in the same way.

9.3 Steps to set a new IF program

9.3.1 Select IF therapeutic mode and program

Press [MODE] button until the LCD display "IF" and expected program (N, M, M1).

9.3.2 Set interference frequency

Press [SET] button to enter setting mode, press [▲] or [▼] button to adjust the parameter from 1Hz to 150Hz.

9.3.3 Set timer

There are two adjustable timers available. The left timer controlling the treatment time of CH1 and CH2. The right timer controlling the treatment time of CH3 and CH4. The treatment time is adjustable from 1 to 60 minutes

or C (Continuous). Press **[SET]** button to enter this menu, then press **[▲]** or **[▼]** button to adjust the setting. Press **[▲]** button when the timer shows 60 minutes, it will be switched to continuous stimulation. Two timers can be set in the same way.

9.4 Steps to set a new RUSS program

9.4.1 Select RUSS therapeutic mode and program

Press **[MODE]** button until the LCD display “RUSS” and expected program (N, M, M1).

9.4.2 Set burst frequency

Press **[SET]** button to enter setting mode, press **[▲]** or **[▼]** button to adjust the burst frequency from 20Hz to 80Hz.

9.4.3 Set timer

There are two adjustable timers available. The left timer controlling the treatment time of CH1 and CH2. The right timer controlling the treatment time of CH3 and CH4. The treatment time is adjustable from 1 to 60 minutes or C (Continuous). Press **[SET]** button to enter this menu, then press **[▲]** or **[▼]** button to adjust the setting. Press **[▲]** button when the timer shows 60 minutes, it will be switched to continuous stimulation. Two timers can be set in the same way.

Chapter 10:

PATIENT COMPLIANCE METER

This unit can store 60 sets of operation records. Total treatment time up to 999 hours can be stored.

Check & Delete Individual Record

Press **[MODE]** button and turn on the power simultaneously. The LCD will show the number of records and operation time. Press the **[▲]** or **[▼]** button to check each record.

To delete a record, press **[SET]** button for 3 seconds.

Check & Delete Accumulative Record

At the individual records menu, press **[MODE]** button to switch to accumulative record menu. Press the **[SET]** button first, then press the **[MODE]** button simultaneously for 3 seconds and all of the records will be deleted followed by a beeper sound.

Chapter 11:

MAINTENANCE AND STORAGE

- 1) Non-flammable cleaning solution is suitable for cleaning the device.
Note: Do not smoke or work with open lights (for example, candles, etc.) when working with flammable liquids.
- 2) Stains and spots can be removed with a cleaning agent.
- 3) Do not submerge the device in liquids or expose it to large amounts of water.
- 4) Return the device to the carrying box to ensure that the unit is well-protected before transportation.
- 5) If the device is not to be used for a long period of time, remove the batteries from the battery compartment (acid may leak from used batteries and damage the device). Put the device and accessories in carrying box and keep it in cool dry place.
- 6) The packed TENS/ EMS device should be stored and transported under the temperature range of $-10^{\circ}\text{C}\sim 50^{\circ}\text{C}$, relative humidity 10%~90%, atmosphere pressure 700 hPa~ 1060 hPa.

Chapter 12: TROUBLESHOOTING

Problem	Possible	Cause
Displays fail to light up	Make sure that the batteries are full and were inserted with the correct polarity	Try new batteries.
		Reinsert batteries, ensure batteries are inserted correctly.
	There is an error in the electronics	Remove the batteries and reinsert them after approx. 3 seconds
Stimulation weak or cannot feel any stimulation	Electrodes Dried out or contaminated	Replace
	Lead wires Old/worn/damaged	Replace
Intermittent output	Lead wires	Verify connection is secure. Insure firmly
		Turn down the intensity. Rotate lead wires in socket 90°. If still intermittent, replace lead wires.
		If still intermittent after replacing lead wires, a component may have failed. Call the repair department.
Intermittent output	Program option in use	Some programs will seem intermittent. This is expected.
		Option Controls in the Operation section for a description of the program option.
Stimulation is uncomfortable	Intensity is too high	Decrease intensity
	Damaged or worn electrodes or lead wires	Replace

Stimulation is uncomfortable	Electrode active area size is too small	Replace electrodes with large more area
	May not be operating the device according to the manual.	Please check the manual before use.
Stimulation is ineffective	Improper electrode and applicator placement	Reposition electrode and applicator
	Unknown	Contact Clinician
The skin becomes red and/or you feel a stabbing pain	Use the electrodes on the same site every time.	Re-position the electrodes. If at any time you feel pain or discomfort stop use immediately.
	The electrodes aren't stuck onto the skin properly.	Ensure the electrode is stuck securely on the skin.
	The electrodes are dirty.	Clean the electrode pads with a damp, lint free cloth or replace new electrode pads.
	The surface of the electrode was scratched.	Replace new electrode.
Output current stops during therapy	The electrode pads come off the skin.	Turn off the device and stick the electrode pad firmly to the skin.
	The cable is disconnected	Turn off the device and connect the cable.
	The power of the batteries has been exhausted.	Please replace them with new batteries.

Chapter 13: NORMALIZED SYMBOLS

	Applied part of type BF
	Electrical devices are recyclable material and should not be disposed of with household waste after their useful life! Help us to protect the environment and save resources and take this device to the appropriate collection points. Please contact the organization which is responsible for waste disposal in your area if you have any questions.
	Refer to instruction manual because of the higher levels of output.
	Keep dry
	Type of protection against electric shock: Class II Equipment

Chapter 14: WARRANTY

Twin Stim® Plus 3rd Edition Combo Stimulator carries a warranty of one year from the date of delivery. The warranty applies to the stimulator only and covers both parts and labor relating thereto.

The warranty does not apply to damage resulting from failure to follow the operating instructions, accidents, abuse, alteration or disassembly by unauthorized personnel.

Manufactured for:



Richmar®

Compass Health Brands Corp.
Toll Free 1,888,549,4945
6753 Engle Road
Middleburg Heights, OH 44130
richmarweb.com