Hot & Cold Pack

 **CAUTION: READ BEFORE USING.**

CAUTIONS (TO REDUCE THE RISK OF ACCIDENT)

- Use additional caution when applying to face, neck, or eyes – skin may be more sensitive to extreme temperatures.
- **DO NOT** use heated on eyes
- Adult supervision is recommended when using on children.
- Consult your doctor before treating any serious injury or when pain persists.

WARNINGS AND DANGERS

- **DO NOT** use the product without protective wrap (if called out in instructions).
- **DO NOT** sit or lay on the product. Doing so may increase risk of temperature-related injury, and may cause the device to leak.
- **DO NOT** use while sleeping or under the influence of mind-altering drugs, alcohol, or while suffering from dizziness.
- **DO NOT** use this product on persons with nerve damage, circulatory problems, impaired skin sensation, or with damaged or broken skin.
- **DO NOT** use this product on persons unable to remove this product such as infants, children, and some older persons.
- **DO NOT** use in conjunction with liniments, balms, lotions or any product containing topical pain relief ingredients.

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